# HICO-AQUATHERM 660

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32,5 x 23 cm  
REF 550086

# HICO-AQUATHERM 660

On the basis of many years of experience our hyperthermia unit HICO-AQUATHERM 660 combines the advantages of various warming mat systems and sets new standards.

- maximum heat transfer by water
- compatible with all HICO-warming mats \*
- connexions for two warming mats
- precise temperature control
- new safety system
- compact and robust construction
- simple handling
- \* see accessories

## Description and mode of operation

**HICO-AQUATHERM 660** was constructed according to the latest trends in the field of normo- and hyperthermia used for patients. The heat transfer to the patient is effected by surface contact. The heat is conducted to the patient by means of a water circuit between aggregate and warming mat. Due to its good heat conductivity and heat capacity, water is an ideal medium for non-electric heat transport.

Owing to its compact construction the new device can be used as table top model or mobile footed unit. It can also be fixed on a wall bar.

In order to provide multiple use of the unit, we offer various types of mats, according to the patient's needs.

**Main Indications:** Anaesthesia, Surgery, Intensiv Care, Paediatrics, Emergency, Veterinary medicine

Fig.1

## Product features

- Handling of **HICO-AQUATHERM 660** is very simple and safe.
- The temperature of the warming mat can be adjusted by push-button from 35-39°C.
- According to the range of application one or two HICO-warming mats (see accessories) of different shape and size can be connected at any time.
- The automatic function test and easy readable displays inform the user about regular functioning of the unit at any time. In case of malfunction, user and patient will be protected by the new safety system.

## Guarantee

The **HICO-AQUATHERM 660** is conform to the MDD 93/42/EEC directive and bears the CE-mark. The unit is manufactured and controlled according to ISO 9001 and IEC 601-1.

We grant a warranty of 24 months in case of any quality defects.

## Product data

Article-No. (REF):	590001
Operating voltage:	230 VAC, 50/60 Hz
Power consumption:	530 W
Setting range for water temperature:	35-39°C
Water content:	abt. 0.7/1.4 l (min/max)
Pumping level:	10 W (19 l/minute max.)
Warming-up time (20-37°C):	abt. 5-10 min.
Safety cutoff:	41.5°C
System of protection:	I, BF
Class of risk (93/42/EEC):	II b
Weight:	abt. 9 kg
Noise rate:	abt. 35 dB(A)
Alarm level:	> 65 dB(A)
Dimensions (W x H x L):	abt. 200 x 290 x 330 mm
Testing basis:	MDD93/42/EEC, IEC 601-1 IEC 601-1-2, IEC 601-2-35

Technical modifications reserved

## Accessories

### Hose extension

abt. 3 m REF 550022

### Mobile footed unit (Fig. 2)

Five-foot rack REF 590002

Clamping holder REF 592900

### Wall bar mounting (Fig. 2a)

Clamping holder REF 592900

### HICO-PATIENT MAT (Fig. 3) "Standard"

- carefree and elastic material
- |             |            |
|-------------|------------|
| 170 x 50 cm | REF 550030 |
| 92 x 50 cm  | REF 550031 |
| 150 x 35 cm | REF 550032 |
| 92 x 35 cm  | REF 550035 |
| 32 x 35 cm  | REF 550033 |

### HICO-AQUASOFT (Fig. 4) with Decubitus prevention

- pressure relief up to 52%
  - soft, flexible and light
- |             |            |
|-------------|------------|
| 100 x 50 cm | REF 550070 |
| 150 x 50 cm | REF 550071 |
| 185 x 85 cm | REF 550072 |

### HICO-PATIENT BLANKET (Fig. 5) to be used under or over the patient

- flexible material, kind to the skin
  - low weight
- |             |            |
|-------------|------------|
| 56 x 70 cm  | REF 550037 |
| 120 x 70 cm | REF 550038 |

### HICO-AQUALIGHT (Fig. 6)

very light  
flexible and soft  
50 x 30 cm

REF 550080

### HICO-SOFT-TEMP (Fig. 7)

Mat/Cover for one-way use  
above and/or below the patient  
low weight, kind to skin

**Set** 5x mat/cover 81 x 49 cm  
+ 1x hose extension REF 550066

**Set** 5x mat/cover 160 x 49 cm  
+ 1x hose extension REF 550065