HICO-VARIOTHERM 550

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HICO-VARIO THERM 550

On the basis of many years of experience in the cooling and heating of patients the hypo-/hyperthermia unit **HICO-VARIO THERM 550** combines the advantages of various hypo-/hyperthermia systems and sets new standards.

- optimum heat transfer or discharge by water
 - compatible with all HICO-water mats *
 - temperature range 15-39 °C
 - very precise temperature control
 - new safety system
 - compact and robust construction
 - simple handling
 - low noise
- * see accessories

Description and mode of operation

HICO-VARIO THERM 550 was constructed according to the latest trends in the field of hypo- and hyperthermia used for patients. The cooling and heating of patients is effected by surface contact. The heat is conducted to the patient by means of a water circuit between aggregate and water mat. Due to its good heat conductivity and heat capacity, water is an ideal medium for non-electric, save and effective heat transport.

Owing to its compact construction the new device can be used as table top model or mobile footed unit.

In order to provide multiple use of the unit, we offer various types of mats, according to the patient's needs.

Product features Fig.1

- Handling of **HICO-VARIO THERM 550** is very simple and safe.
- The temperature of the water mat can be adjusted by push-button from 15-39°C.
- According to the range of application **HICO-water mats** of different shape and size can be connected.
- The automatic function test and easy readable displays inform the user about regular functioning of the unit at any time.

Warrenty

The **HICO-VARIO THERM 550** is conform to the MDD 93/42/EEC directive and bears the CE-mark. The unit is manufactured and controlled according to ISO 9001 and IEC 601-1.

We grant a warrenty of 24 months in case of any quality defects.

Product data

Article-No. (REF):	540001
Operating voltage:	220-230 VAC/50 Hz
Power consumption:	320 W
Setting range for water temperature:	15-39°C
Water content:	abt. 0.5/1.0 l (min/max)
Pumping level:	max. 11 l/minute, max. 0.15 bar
Cooling time (20-15):	abt. 5-10 min.
Warming-up time (20-37°C):	abt. 5-10 min.
Safety cutoff:	41.5°C
System of protection:	I, BF
Class of risk (93/42/EEC):	II b
Weight:	abt. 17 kg
Noise rate:	abt. 50 dB(A)
Alarm level:	> 65 dB(A)
Dimensions (W x H x L):	abt. 200 x 290 x 440 mm
Testing basis:	MDD93/42/EEC, IEC 601-1 IEC 601-1-2, IEC 601-2-35

Technical modifications reserved

Accessories

Hose extension

abt. 3 m REF 550022

Mobile footed unit (Fig. 2)

Five-foot rack REF 530002
incl. Supporting plate

HICO-PATIENT MAT (Fig. 3) "Standard"

- carefree and elastic material
- | | |
|-------------|------------|
| 170 x 50 cm | REF 550030 |
| 92 x 50 cm | REF 550031 |
| 150 x 35 cm | REF 550032 |
| 92 x 35 cm | REF 550035 |
| 32 x 35 cm | REF 550033 |

HICO-AQUASOFT (Fig. 4) with Decubitus prevention

- pressure relief up to 52%
 - soft, flexible and light
- | | |
|-------------|------------|
| 100 x 50 cm | REF 550070 |
| 150 x 50 cm | REF 550071 |
| 185 x 85 cm | REF 550072 |

HICO-AQUALIGHT (Fig. 6)

very light
flexible and soft
50 x 30 cm
REF 550080

HICO-PATIENT BLANKET (Fig. 5) to be used under or over the patient

- flexible material, kind to the skin
 - low weight
- | | |
|-------------|------------|
| 56 x 70 cm | REF 550037 |
| 120 x 70 cm | REF 550038 |

HICO-SOFT-TEMP (Fig. 7)

Mat/Cover for one-way use
above and/or below the patient
low weight, kind to skin

Set 5x mat/cover 81 x 49 cm
+ 1x hose extension REF 550066

Set 5x mat/cover 160 x 49 cm
+ 1x hose extension REF 550065